

Ceiling Rose Pendant

SPECIFICATION SHEET

The MEC Ceiling Rose Pendant is available in seven different configurations ranging from one pendant gas assembly to six assemblies (seven with AGSS added). Gas types and hose lengths must be specified on ordering.

Features

- A combination of up to seven gases (six plus AGSS)
- Installation allowing complete flexibility
- 1m coloured gas specific anti-static hose
- (MRI version available, not as standard)
- Phthalate Free
- Manufactured to current industry standards
- Anti-Microbial.

Applications

- Convenient delivery of hospital piped gases.

Specification

Dimensions of Ceiling Cowl:

Diameter:	235 mm
Depth:	65 mm

Order Codes

One Gas Ceiling Pendant Complete	163631
Two Gas Ceiling Pendant Complete	163632
Three Gas Ceiling Pendant Complete	163633
Four Gas Ceiling Pendant Complete	163634
Five Gas Ceiling Pendant Complete	163635
Six Gas Ceiling Pendant Complete	163636



Supplied by

Ceiling NIST and Copper Tube to connect with hospital pipeline in ceiling cavity.

Ceiling Rose Pendant Assembly

Nut secures each Ceiling NIST and Copper Tube through its hole in the Base Plate.

Ceiling NIST (Female) with Copper Tube.

Base plate is screwed to ceiling using appropriate screws(A) x4 (Not supplied) suitable for the ceiling material involved. **This should be accomplished only after all Ceiling NISTs and Tubes have been secured to the Base Plate.**

Rubber Seal is put around edge of Cowl before attachment to Base Plate.

Cowl locates over Base Plate and is secured by two Screws (B).

Rubber Grommets supplied to blank unused holes in Cowl.

Screws (x2), locate through the Cowl and into pre-threaded holes on the Base Plate.

Each **Pendant Hose Assembly** screws into the projecting threaded outlet from the appropriate Ceiling NIST (Female) for each gas. (O2 shown as example)

Up to seven Pendant Hose Assemblies can be fitted. Any unused holes can be blanked using Rubber Grommets (supplied). Length and types of Pendant Hose Assemblies need to be specified at time of ordering.

Please note that appropriate fixings must be used to attach the Base Plate to the ceiling. Please refer to ceiling manufacturer's guidelines for recommended fixings when used with their product.

Cowl and Base Plate shown from below.



Supplied by