

Dial Flowmeter

SPECIFICATION SHEET


MEC Dial Flowmeters are designed and built to the highest standards with clear display of the flow rate.

Features


- Oxygen Dial Click Flowmeter with British Standard Probe
- Choice of type of output fitting. Therapy Barb, DISS 9/16"
- Choice of number of output fittings. (1 or 2)
- Choice of oxygen therapy outlet with multiple click-stop selector
- Multiple flow therapy output 0 ½, 1, 2, 3, 4, 6, 8, 10 and 15 litres per minute.

Specification

Weight	0.2Kg
Height	6.0cm
Depth	6.0cm
Width	8.0cm

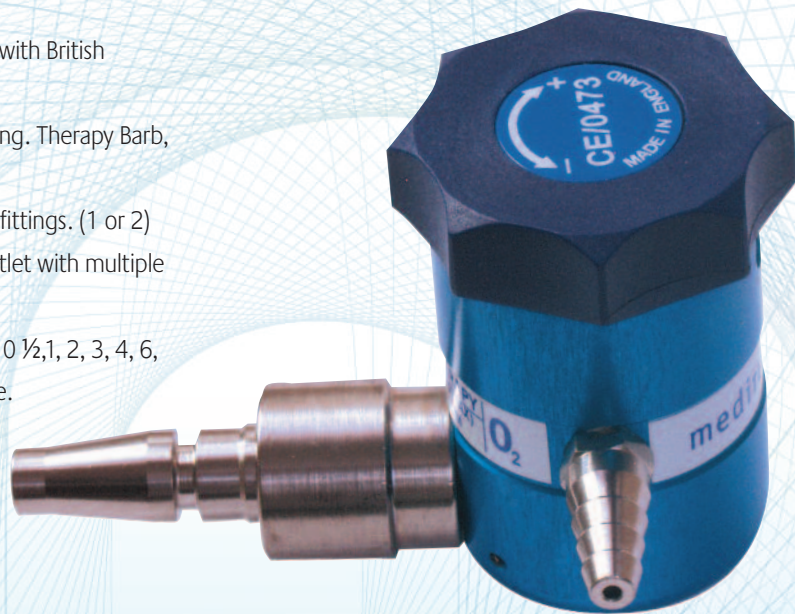

O2 0.5 – 15Lpm Flowmeter on fixed probe.

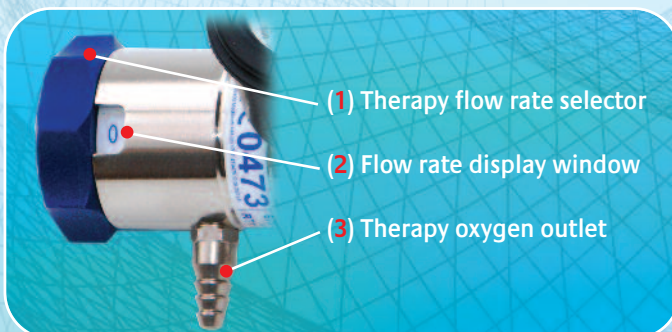

Oxygen therapy is easily chosen by rotating the "Therapy flow rate selector" (1). The flow rate is indicated in the display window (2). Choose from ½, 1, 2, 3, 4, 6, 8, 10 and 15 litres per minute. Oxygen at the chosen flow is discharged from the Therapy outlet (3). The Therapy outlet is designed so that plastic oxygen tubing can be securely held in place on the fir-tree style connector.

Note: The flowmeters are designed so that the flow rate display window can be easily seen, at the same time the therapy outlet can be accessed so that equipment may be attached.



Supplied by